# ETO C 1445

## ETİLEN OKSİT STERİLİZASYON CİHAZLARI (ETHYLENE OXIDE STERILIZATION DEVICES)

**VAKUM ALTI AKTİF, HIZLI, KURU SİSTEM / EO STERİLİZASYONU**  
**(ACTIVE, QUICK AND DRY SYSTEM WITH SUBVACUUM TECHNOLOGY / EO STERILIZATION)**

## AKTİF, HIZLI, KURU SİSTEM

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## ACTIVE, QUICK, DRY SYSTEM

ETO C 1445 Ethylene Oxide Sterilization device sterilizes all kinds of medical, plastic, organic, electromechanical and surgical instruments and also re-sterilizes disposable materials.

### ECONOMICAL AND LONG LIFE STERILIZATION AT DESIRED DIMENSIONS AND CAPACITY

## GENERAL FEATURES

Having microprocessor technology, ETO C 1445 uses silicon heater, cold steam and dry system vacuum sub Ethylene Oxide Sterilization technique. All stages of the sterilization process are carried out in a single sterilization chamber. (all in 1 technology)

ETO C 1445 Ethylene Oxide Sterilization device is programmed to be ready for use at any time with fixed and adjustable between 37°C -55°C ( $\pm 1^\circ\text{C}$ ). During economic working period of 4.5 to 9 hours, the system; automatically detects the values of pressure, vacuum, gas, humidity, air washing and continues the sterilization phases (vacuum-heating-humidification-sterilization-air washing) without any external intervention. Standart aeration time is 30 minutes. Sterilization duration and number of aeration can be adjusted between 1-99 from the user panel.

Sterilized materials can be used directly without any need for extra ventilation.

ETO C 1445 Ethylene Oxide Device uses standart 100gr 100% EO cartridge.

Gas concentration amount inside sterilization chamber during EO exposure process is  $740 \pm 10$  Mg/Lt.

Cartridge is placed by the user into the cartridge slot. It is automatically drilled once the system gets started so the user is prevented from EO exposition.

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Gas concentration amount is detected by the sensors located in the cartridge slot and in the case of filling deficiency, system displays a warning on the screen and does not allow the device to start the sterilization process.

In order to prevent EO exposure, sterilization and air washing processes are done in the same chamber. Also sterilization phases could be performed manually depending on the User requirement. System automatically gives error in case of empty or missing cartridges,

Sterilization chamber design and programme of ETO C 1445 provides safe sterilization for the materials to be sterilized especially for cardiovascular surgical or other laparoscopic type surgical instruments having tube structure without any deformation.

There is Bacteria filter in the ETO C 1445 device and uses the air taken in the sterilization chamber by filtrating through HEPA filter.

## ELECTRONICAL EQUIPMENT AND OPERATING PROGRAMME

There is set up (calibration) settings available in the device.

There are 2 automatic programme modes in device consisting of Normal and Economic programme. Second mode (37°C, 55°C) contains pressure-temperature-time changing programme and vacuum leakage test programme. (leakage detection)

For security reasons, chamber is under vacuum during sterilization and pressure of the sterilization chamber is set to be min – 100 mBar from the ambient pressure during

ETO C 1445 does not need stable heating value. Device works between 37°C-55°C according to the stable or changeable pressure, time and humidity diagram.

Upon closing the door of the device and completion of automatic conditioning at required pressure, humidity and temperature, cartridge is automatically pierced and eo gas is discharged into the sterilization chamber.

Device proceeds operation by controlling all phases. In case of any problem, it doesn't get through to the next phase and gives error.

ETO C 1445 may be operated manually if requested by the user or in case of a failure and device can perform automatic or manual air washing in numbers determined by the user. As soon as the sterilization is completed, air washing is going on automatically until the door is opened to prevent EO gas collection on the sterilized products in the chamber .

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Since microprocessor & PLC technology is used in the device, there is no keyboard. Having touchscreen and digital control panel, in ETO C 1445, all sterilization phases can be graphically monitored on screen touch screen as well as alphanumeric sterilization start/end date-times, heat-pressure-humidity values, sterilization phases (vacuuming, moisturizing, sterilization, air washing) and passing times, program selection, number of sterilization, error, failure, alert, maintenance messages. can be read through the screen, set values can be entered. At the end of sterilization, these data can be recorded by printing from the printer on the device. At the end of sterilization, these values can be taken out from the printer on the device or saved in pdf format with the help of USB.

All hardware can be controlled automatically and manually from the screen. At the end of the sterilization, system warns the user graphically, in writing and audibly, and again in writing with the printout. At the end of sterilization, device warns the user with a graphical, written and audible printout on the screen.

Function features can be monitored digitally in the ETO C 1445. It keeps all the information in its memory in case of any power cuts and continues or restarts the sterilization according to the stage where the program is interrupted and the values inside the sterilization chamber. The user is informed with error codes on the printer output.

Depending on the purpose of use, all existing data values can be changed within 20% tolerances (temperature, humidity, pressure, vacuum) with the detail programming system.

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## FEATURES OF THE EQUIPMENT USED ON THE DEVICE

- ▶ Vacuum pump sucks up to -700mBar. It is an oil pump. In addition to this vacuum can also be provided with compressed air injector system.
- ▶ Components which could cause service problems in the device is at minimum level. There is no equipment such as circulation engine, water heater, steam generator.
- ▶ Sterilization chamber and jacket are made of stainless steel or optionally unpainted Anadize Aluminum. Door component is 316 L stainless steel or anodized aluminum. Exterior parts can be manufactured from 304 L stainless steel or coated with enamel paint or epoxy paint. In ETO C 1445, EO gas resistant and specially designed silicone and rubber door gaskets are used and sterilization chamber insulation is provided with isopolymer material.
- ▶ ETO C 1445 sterilization chamber is resistant to -800 mBar operating pressure.

## NEUTRALIZATION

Increasing the sterilization reliability and rehabilitation of toxic waste is called neutralization. Neutralization is important for preventing environmental pollution, increasing personnel occupational safety, and creating the opportunity to evaluate waste products.

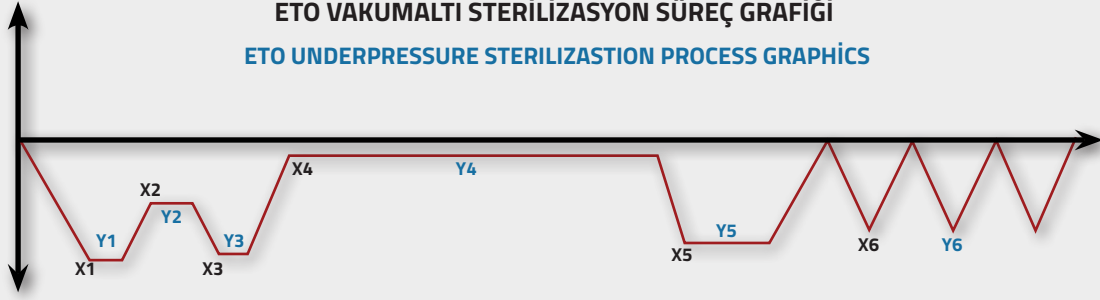
In ETO C 1445, after the sterilization process, evacuation of EO gas in the chamber is done by neutralizing by special ventilation mechanism, and filtering from the exhaust to the atmosphere in a purified state. During this process, water drainage and pump system are not used and drainage is not required.

Ethylene oxide gas is a water-soluble gas.

Together with the neutralized liquid ( $H_2O+H_2SO_4 +b(+)$ ) +(HCl vapor) it turns into waste ethyl glycol (antifreeze liquid). Only  $CO_2$  is thrown out.

ETO C 1445 dissolves EO gas completely (99.9%) in the microprocessor-controlled neutralization unit, and non-toxic gas ( $CO_2$ ) is supplied to the drain or exhaust.

## ETO VAKUMALTI STERİLİZASYON SÜREÇ GRAFİĞİ ETO UNDERPRESSURE STERILIZATION PROCESS GRAPHICS

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## HUMIDIFICATION

It makes humidification from its own tank (humidity and water tank) with distilled water. There is no need for an extra moisturizing package.

It does not need an external dosimeter.

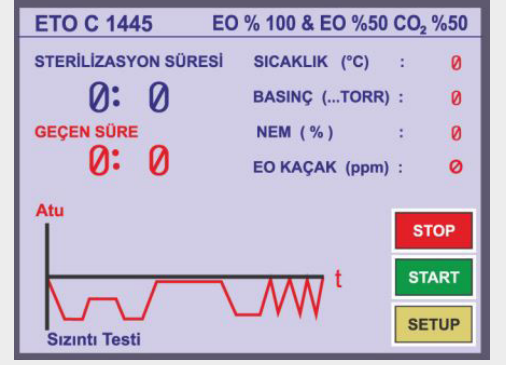
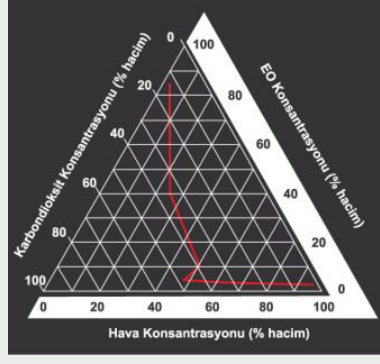
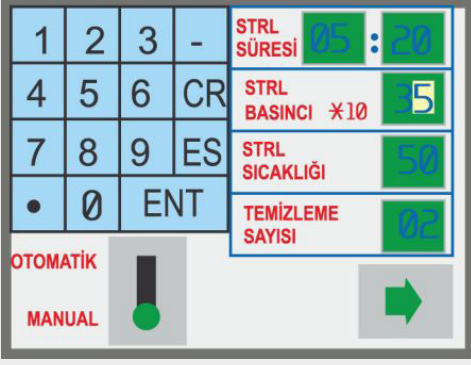
Since the device has a dry cold steam system, electromechanical equipment can also be sterilized.

There is no water condensation inside the packages of the sterilized materials, and there is also no ethyl glycol stain left on the material.

When the humidity level in the device falls below the required value, the user is warned with an automatic alarm system.

Water level in the humidification tank is enough for approximately 15-20 sterilization cycles.

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## HEATING

Dry, economical, homogeneous and maintenance-free sterilization chamber heating is done in a short time with silicone heaters.

There is homogenous heat distribution inside the sterilization chamber owing to private heating system of the device.

## SAFETY AND DOOR SYSTEM

Electronic system of ETO C 1445 has an integrated gas leakage sensor which controls gas leakage, and device warns the user against gas leaks. Safety arrangements of the device prevents the possibility of untimely piercing of the cartridge.

In case of any error, device gives alert visually and audibly. Error type is shown on the screen and also seen with the help of printout.

For safety purposes, there is an hermetic aspirator which can be connected to the manhole.

Door lock system in the device is pneumatic and electronically controlled with microprocessor.

Process never starts until the door is closed. There is also visual icon on the screen showing whether the door is open or not.

Before the process is finished or while there is gas in the sterilization chamber or in case of power cuts, door can not be opened until the system becomes safe.

Ongoing process could be terminated with emergency stop button in the event of any negative situation. In this case, existing EO gas inside sterilization chamber is automatically released to neutralization tank. EO gas is neutralized and exhausted with the help of manhole connection. Once atmospheric pressure is balanced, door can be opened.

No any physical barriers is required to prevent the user from the EO gas at the end of sterilization since the door cannot be opened while there is EO gas in the sterilization chamber.

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Sterilization is performed under vacuum for environment and user safety.

ETO C 1445 Ethylene Oxide Sterilization devices are produced in accordance with 2007/47/EEC and 93/42/EEC, comply with EN and DIN norms, and have CE certificate.

ETO C 1445 is designed and manufactured in accordance with TS-EN 1422 and TS-EN ISO 11135-1 standards, which are specially arranged for ethylene oxide sterilization devices. Device is presented to user after the performance of necessary tests relevant to standards and regulations (EN 60601-1, EN 60601-1-2, EN 62304, EN 1422, EN 62366-1, EN 14971, EN 15223-1)

## ELECTRICAL CHARACTERISTICS

220 V ± 10, 50 Hz, 2,2 KW

## OPTIONAL FEATURES

- ▶ Manhole connection is scaled to user field (min 6mm).
- ▶ Sterilization chamber with Shelf system and stainless steel shelves.
- ▶ Upon request, device could be supplied as tabletop or as footed.
- ▶ Possibility to realize the control connect the control panel to the desired .
- ▶ Remote access to information with RS 232 system
- ▶ Possibility to define authorised user with password.
- ▶ System is suitable to work with externally used detectors.

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